SYNA

— Needle Free Injector —

User Manual


Model : GSP-2000

SYNA

— Needle Free Injector —

User Manual

Model : GSP-2000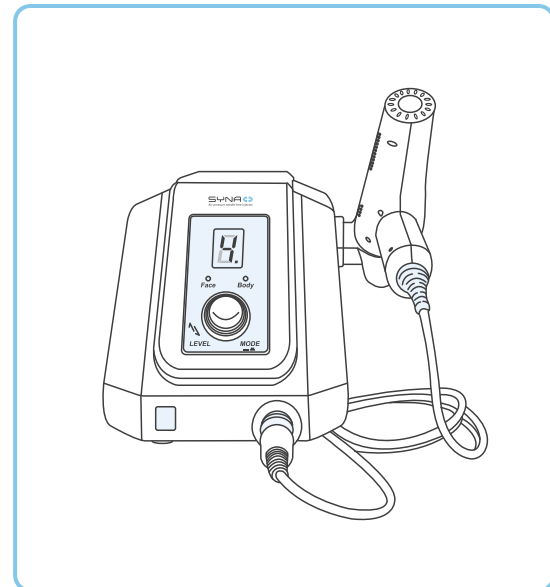

GCS
We make the healthy beauty together
Healthcare

GCS
We make the healthy beauty together
Healthcare

User Manual

For safe use, be sure to read this manual before use, and use it correctly according to the contents described. Also, please keep this manual for future use.
When you transfer this product to someone else, be sure to hand it over with this manual.

* This product is not a medical device, so it cannot be used for medical and drug injections.


Table of Contents

1. Product Composition and Names | 03
2. How to Use | 04
3. Cautions | 07
 - Product-Related Cautions
 - Precautions for Use
4. Product Specifications | 08
5. Service and Support | 08
6. Warranty | 09

1 Product Composition and Names

① Main Body 1ea	② Gun 1ea
③ Power Cable 1ea	④ Manual

① Main Body

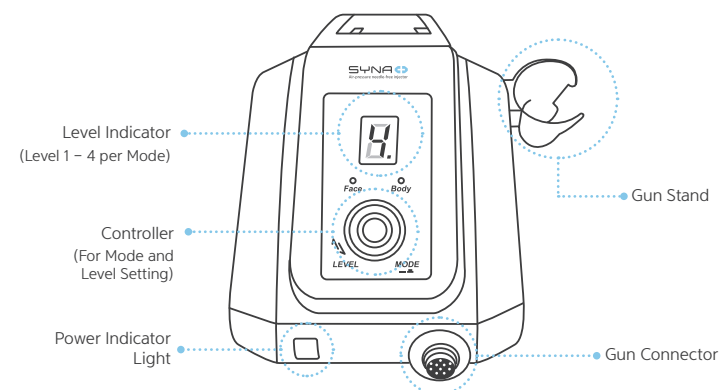② Gun


③ Power Cable

④ Manual


2 How to Use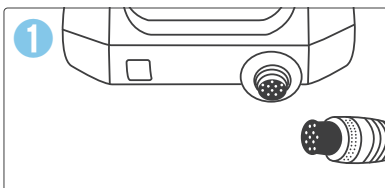


Gun Connection

Before using the device, connect the Gun line to the front of Gun Connector of the Main Body.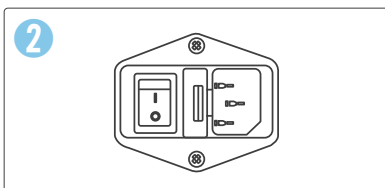


Power Connection

Connect the Power Cable to the back of Main Body, then turn on the power switch to supply power to the device.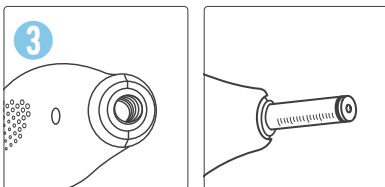


Syringe Connection

After filling the ampoule in the 0.5cc Syringe, connect the Syringe to the front part of the Gun by turning it clockwise as shown in the picture. (Refer to 2-1)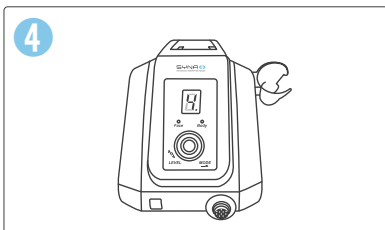

Mode / Level Selection

Press the Controller on the Main Body to select the desired mode (Face / Body). After selecting the mode, turn the Controller left or right to select the desired level. The level can be adjusted from 1 to 4 for each mode. (Refer to 2-2)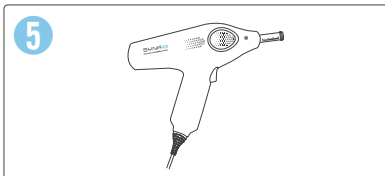


Shot Operation


Put the nozzle part of the Syringe to the skin area you want and press the Gun Operation Button. (Refer to 2-3)

2-1 How to use the Syringe and Optional accessories (sold separately)

■ Optional accessories (sold separately)

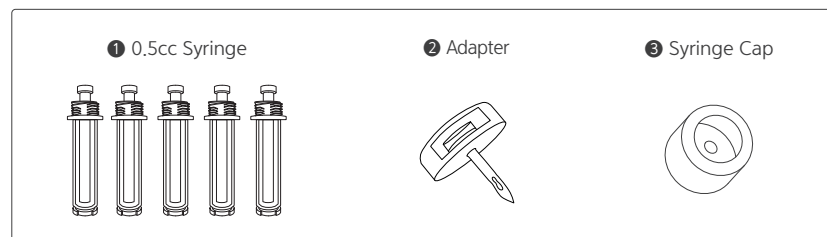
- ① Attach the Adapter to the cap of the vial.
Syringe + Adapter + Vial Assembly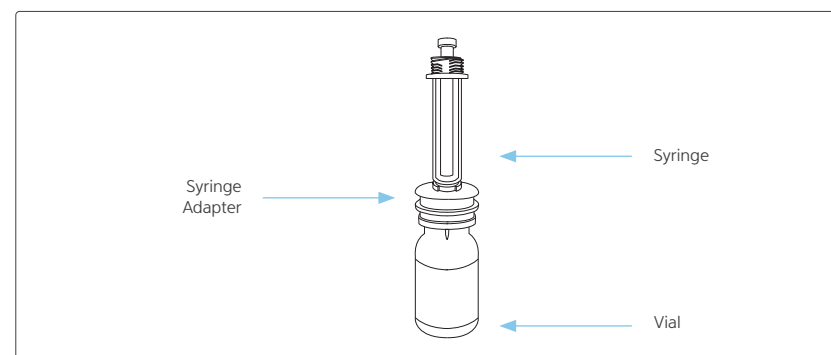


- ② Connect the Syringe to the Adapter and fill the ampoule to the required quantity (0.5cc or less). Before connecting to the Gun, check if there is air in the Syringe, and if there is, push the Syringe plunger to remove the air.
- ③ Connect the filled Syringe to the Gun by turning it clockwise.
- ④ When performing a shot, it is recommended to put the Syringe Cap on the Syringe nozzle to improve skin adhesion and prevent ampoule leakage.

■ Assembly of Syringe · Syringe Cap · Gun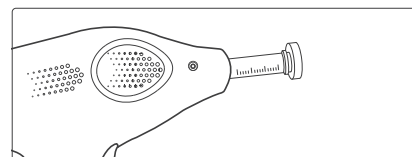

2-2 How to Use the Controller

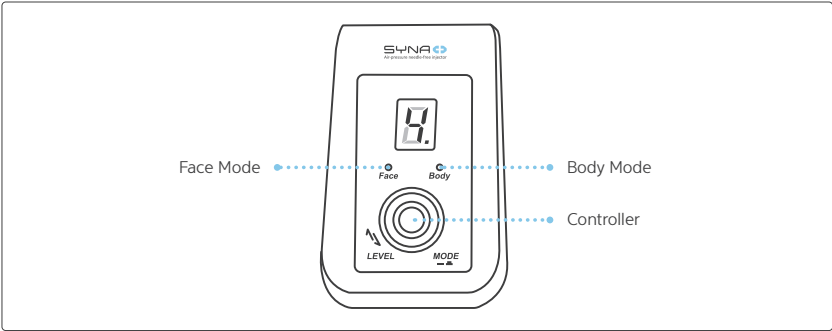- ① Lightly press the center of the Controller button to set the desired mode (Face / Body).
- ② Turn the Controller left and right to set the level (1 – 4) for each mode. (Left: Decrease Level / Right: Increase Level)

2-3 How to Use the Gun · Syringe

- ① When you press the Gun Operation Button, the ampoule is discharged. (1 shot / 1 operation)
- ② Put the Syringe at a right angle (90 degree) on the skin, then press the Gun Operation Button.
- ③ When the ampoule is used up, replace it with a filled Syringe.
- ④ If some residue is not injected, increase the one level.

■ Recommended usage level for each mode and part.

Face Mode Level 1 – 4		Body Mode Level 1 – 4	
Level 1	Eyelids, Eye Bags & Dark Circle Forehead	Level 1-2	Detox and Pain Relief – Whole body (Neck, Shoulders, Back, Arms, Legs, Knees, Abdomen, Vulva)
Level 2	Forehead, Crow's feet, Nose Tip & Bridge, Temple	Level 3-4	Body Contouring – Whole body (Upper Arms, Back, Abdomen, Flanks, Thighs etc.)
Level 3	Chin, Cheeks, Nasolabial Folds		
Level 4	Double chin, Square Jaw, Neck and Other Thick Skin Areas		

※ The usage area for each level may vary depending on skin types.

3 Cautions

■ Cautions about Product

- ① To avoid overheating, allow the product to rest for 5–10 minutes after an hour of continuous use.
- ② The Syringe is for single use only. Please do not reuse.
- ③ Make sure the Syringe put the skin closely before shot operation. Ensure there is no gap between the Syringe and your skin.
- ④ If the Syringe is pressed too firmly against the skin, it may not discharge due to air pressure interference. This is not a defect. Adjust the contact pressure and try shot again.
- ⑤ For your safety, do not attempt to disassemble, repair, or alter the product. This may lead to fire, electric shock, or injury due to malfunction.
- ⑥ Handle the product with care to prevent any accidental drops or impact that could damage it.
- ⑦ When unplugging the device, grasp the plug itself rather than the cord to prevent potential injury or damage to the device.
- ⑧ When the device is not in use for a long term, ensure it is kept powered off.
- ⑨ Ensure children do not use this product.

■ Precautions for Use

- ① You may experience some redness or tiny blood spots in the treated area, but these should fade within a week, depending on individual health condition.
- ② On the day of treatment, avoid applying makeup to your face. If you need to go out, please make sure to apply sunscreen.
- ③ After treatment, stay away from saunas and excessive exercise.
- ④ This product is not suitable for who has keloid skin, thin skin, skin allergies, skin diseases, hypertension, or diabetes.
- ⑤ If you're currently undergoing radiation therapy or have recently undergone surgery and haven't fully recovered, the use of this device isn't recommended.
- ⑥ After treatment, apply a soothing mask or regenerative cream to the treated area, ensuring to maintain the area clean.

4 Product Specifications

- Product Name: SYNAPLUS
- Main Function: Solenoid
- Power: 100~240V
- Power Consumption: 50W
- Packaging Unit: 1 SET
- Weight: 3.0kg
- Size: 160*330*160mm
- Country of Manufacture: South Korea
- Manufacturer: GCS Co., Ltd.

5 Service and Support

- ① Warranty Policy :
Replacement within 7 days after receipt, free repair for 1 year.
(excluding damages caused by misuse)
- ② The following cases require a paid repair :
 - ① Failure due to power supply issues or faulty connection devices.
 - ② Failure due to unauthorized repairs, structural alterations, or performance modifications.
 - ③ Cases of product damage or destruction caused by user negligence.
- ③ Customers are responsible for shipping costs incurred by paid repairs.

6 Warranty

- This product is guaranteed for quality for 1 year from the date of purchase.
- If the warranty period cannot be verified, the warranty period will apply from 3 months after the manufacturing date.
- Defects caused by the product's production process will be repaired or exchanged free of charge, depending on the condition.
- Damages caused by user negligence are not covered by free repairs, even within the warranty period.
- Repairs performed outside of our designated service centers are not covered by this quality assurance.
- The service period is not extended after either free or paid repairs.

Product Name		SYNAPLUS		Model	GSP-2000
Serial No.(S/N)					
Place of Purchase				Tel. No. of the Place of Purchase	
Date of Purchase					
Customer Informatio	Name			Tel. No.	
	Address				

* GCS Co., Ltd. products can be exchanged or compensated in accordance with the consumer dispute resolution standards set by the Fair Trade Commission.

Room 301 - 307, 33, Sagimakgol-ro 62beon-gil, Jungwon-gu, Seongnam-si,
Gyeonggi-do, Republic of Korea
(Sangdaewon-dong, Seongnam Center-M Knowledge Industry Center)
Tel : 02 - 555 - 1536 | Fax : 02 - 3142 - 7670
Service Center : 070 - 4350 - 6190 | Service Request: gcscv.co.kr

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